




Function Control Module (Relay)

ZKRUNCIPRMLK100




Enables on/off control for two different functions within the room (such as reading lamp or window blinds) from the patient handset. One Function Control Module must be installed per Patient Handset, connected between the Room Control Unit and the Bedside Call Unit via its two RJ45 ports (in/out), or connected to a Call Unit port on the Room Control Unit by itself.

The lamps take orders from the Room Control Unit so they must be assigned to the handset which will give them orders. This process takes place after the Call Units are given addresses.

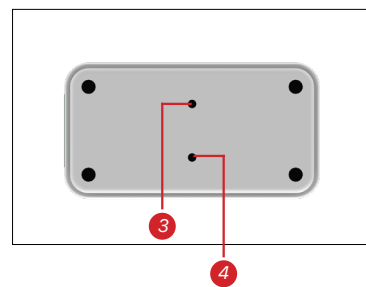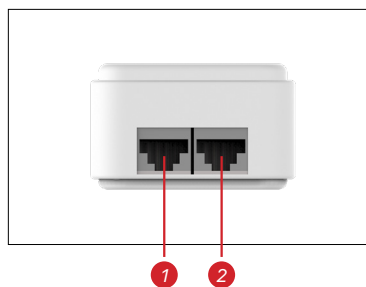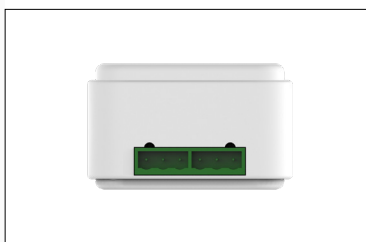

1. Data Out
2. Data In
3. Config Button
4. Status LED

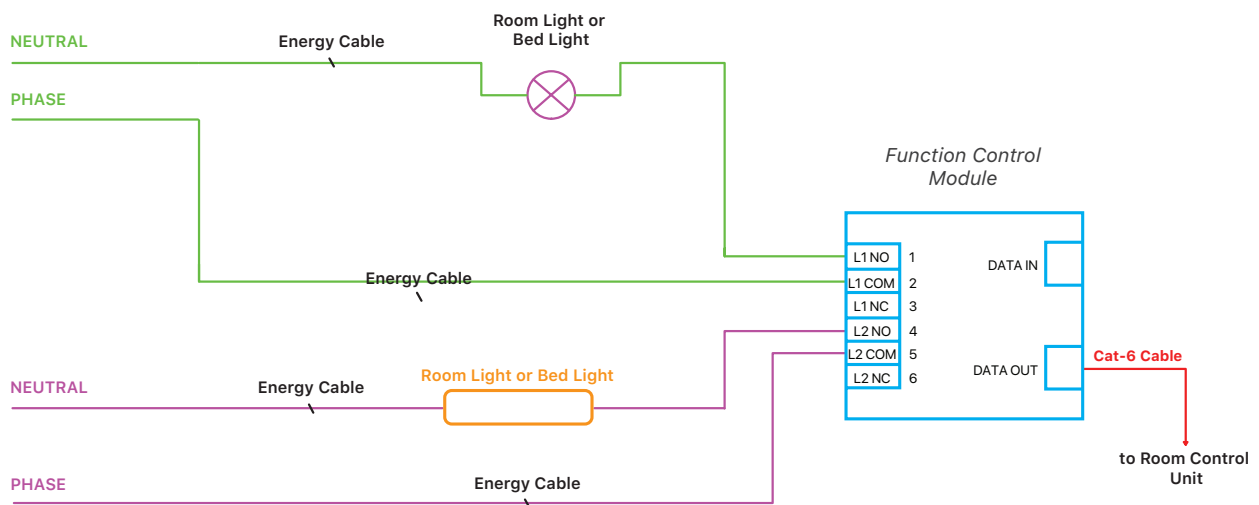

Technical Specification

ZKRUNCIPRMLK100

Weight	75 g
Width	80 mm
Depth	50 mm
Height	40 mm
Operating Temperature / Humidity	-20°C +60°C / % 30 %80 RH
Supported Light Control Types	Vaviem or Dry Contact
Ports and Slots	2xNo-com-NC, 2xRJ45
Color	Gray